1

2

- ▲ Use only sterile saline solution!
- Uporabite samo sterilno fiziološko raztopino!
- Koristite samo sterilnu fiziološko otopinu!

3

4

EN Mediblink Nasal shower M418

Intended Use/User: Used for flushing the patient's nose or for flushing between different drug treatments. For single use only. Users are patients or healthy people including children and babies older than 4 Months with adult supervision.

Instruction for Use: Open the packaging, take out the syringe and soft silicone nozzles, draw physiological saline into the syringe, and install the soft silicone nozzles. Place the soft silicone nozzle into the nasal cavity, **tilt the head slightly forward**, open the mouth, and push in physiological saline solution. After rinsing, use a tissue to wipe off any spilled liquid. **Composition:** PP plastic and silicone. **Content:** 2 syringes, 2 soft silicone nozzles.

Contraindications: Patients with damage or abnormalities in the nasal cavity or sinuses; Patients with nasal bleeding; Patients who have allergic reactions to cleaning solutions or cleaning agents; Prohibited for children under two weeks old; Use with caution with patients with mental illness or severe intellectual disability; Patients with severe hemoptysis or respiratory obstruction who cannot tolerate it due to systemic diseases; People who are allergic to product materials must caution use.

▲ **Warnings:** Store in a cool, dry place. The solution temperature should be close to body temperature (must not exceed 40 degrees Celsius). Thoroughly clean the nasal shower with clean water before first use. Do not use it when one nostril is completely blocked or when there is an ear infection. It should not be used after drinking alcohol or by those who are unable to take care of themselves. Do not rinse the nasal cavity directly with purified water, otherwise it will stimulate the nasal cavity and cause discomfort. A Disposable device (for single patient use only). Non sterile product. Does not contain rinsing solution. If severe injury is present, please seek medical advice prior to application. If any serious incident has occurred in relation to the device, it should be reported to the Manufacturer and the Competent Authority of the Member State in which the user and/or patient is established.

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Ningbo Greetmed Medical Instruments Co., Ltd.
MedPath GmbH