Mediblink®
medical devices

EN Mediblink Electric nasal aspirator SuperEasy M405

INSTRUCTIONS FOR USE

For correct use, please make sure you have read this user manual.

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TABLE OF CONTENTS

1. Foreword	6
2. Declaration of Conformity	6
3. Technical Data	6
4. Part Identifications	7
4.1 Basic Components	7
4.2 Display	7
5. How to Use	7
6. Installation and Operation Instructions	8
6.1 Installation Parts Description	8
6.2 How to Install	8
6.3 Before Use	9
6.4 Installing and Replacing the Batteries	10
6.5 Operating Instructions	10
7. Troubleshooting	13
8. Care and Maintenance	14
8.1 Product Maintenance and Storage	14
8.2 Storage and Transportation Conditions	14
9. Product Type	14
9.1 Intended Use	14
9.2 Lifespan	14
10. Important Safety Information	15
11. Package	18
12. Disposal	18
13. EMC Guidance	18
14. Warranty	19

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1. Foreword

Thank you for choosing the Mediblink Electric nasal aspirator SuperEasy. This product is used to clean a baby's nose. It is intended mainly for home use.

This product is made with precise components; please keep it away from high temperature, humid environments, sunlight, shake and dust. For correct use, please make sure you have read this user manual. Please keep it after reading, in case you need to refer to it again. For personal hygiene use only. This is NOT a medical product. If symptoms are severe or persistent, consult your doctor.

2. Declaration of Conformity

- This product is designed in accordance with the Standard EN 60335 for household and similar electrical appliances.
- This product has been tested and approved in accordance with EN IEC 55014 (Electromagnetic compatibility – Requirements for household appliances, electric tools and similar apparatus) and EN IEC 61000 (Electromagnetic compatibility – EMC). However, this does not guarantee that the product will not be affected by electromagnetic interference. Avoid using the product in environments with high electromagnetic disturbances.


3. Technical Data

Name	Electric nasal aspirator
Model / REF	H2606EU / M405
Display	LED
Maximum suction force	60 kPa (8.70 PSI) ≤ absolute value ≤ 70 kPa (10.15 PSI)
3 suction levels	AUTO I II
Automatic power-off	≤ 65 s
Noise	≤ 60 dB(A)
Power source	DC 4.5 V, 3 AAA 1.5 V alkaline batteries
Life span	5 years
Dimensions	6.89 x 2.68 x 1.65 inches / 17.5 x 6.8 x 4.2 cm (L x W x H)
Product weight	156 g ± 10 g (including batteries)
Operating conditions	Ambient temperature: 5 °C–40 °C
	Relative humidity (RH): ≤ 85 %
	Atmospheric pressure: 62 kPa–106 kPa
Transport and storage conditions	Ambient temperature: -20 °C–55 °C
	Relative humidity (RH): ≤ 93 %
	Atmospheric pressure: 62 kPa–106 kPa

4. Parts Identification


4.1 Basic Components

1. LED Screen
2. Power button (long press to switch on/off, short press for gear shift)
3. Plastic shell case
4. AAA batteries
5. Battery door
6. BPA-free suction nozzle
7. Transparent cover
8. Silicone cover
9. Air inlet of vacuum pump
10. Collecting cup
11. Sealing ring of collecting cup
12. Collecting cup base
13. Storage compartment base sealing ring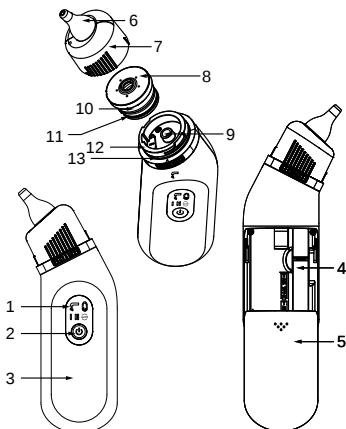

4.2 Display

14. Cleaning prompt
15. Battery power indicator
16. Suction level indicator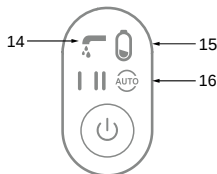

5. How to Use

1. Make sure your hands and removable parts of the device are sanitized, and that all parts are snapped into their correct place before use.
2. Soften stubborn pieces of mucus with saline and wait for 30–60 seconds; or apply a hot towel on the baby's nose for 10 min and massage; or wet with a cotton swab dipped in warm saline for 5–10 min.
3. Choose proper suction nozzles and gear to suction mucus. Advice: make sure the baby's mouth is open to avoid negative pressure entering the ear and mouth.
4. After use, power it off and remove the transparent cover with suction nozzle and the cup from the main body, and clean with warm water (neutral detergent is suitable if needed), you can also clear the suction nozzle with a straw brush and then air it dry.

6. Installation and Operation Instructions

6.1 Installation Parts Description

	Wide nasal suction nozzle (for ages > 2 years)		Transparent cover
	Narrow nasal suction nozzle (for ages ≤ 2 years)		Collecting cup
	Sealing rings		Silicone cover

6.2 How to Install

Before Installation


Before installation, please check whether the product appearance is good and complete. If there is any defect, please contact the supplier or manufacturer as soon as possible.

Component Installation


- a) Gently fit the small sealing ring into the groove at the bottom of the collecting cup.


- b) Put the silicone cover on the collecting cup, then gently press around the silicone cover to ensure it is properly installed.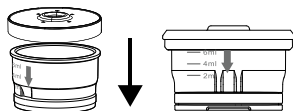

- c) Align the collecting cup with the gap of the cup base and install it onto the body.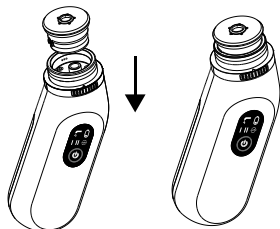

- d) Press the transparent cover on the body.

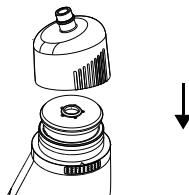- e) Install the BPA-free suction nozzle on the transparent cover.

Note: Keep fingers away from the suction nozzle during installation to avoid bacterial contamination.

6.3 Before Use

- a) If the user's nose is dry, it should be humidified first.

Apply a hot towel to both sides of the baby's nose for about 10 minutes. When the nasal mucosa contracts with heat, the nasal cavity becomes smooth, and the viscous nasal mucus will also be hydrated and more easily absorbed. Or moisten the inner dry knot of the baby's nose with a cotton swab dipped in warm physiological saline/sesame oil for about 5–10 minutes and then suction it out.

Do not use force or pull harshly when suctioning dried mucus from the baby's nose.

- b) Cleaning and disinfection

Before use, ensure that the BPA-free suction nozzle is clean and sanitized.

- c) Before use, check whether the suction of the BPA-free suction nozzle is working properly. If there is air leakage, verify that the transparent cover is snapped in place, and that the sealing ring at the bottom is installed and fits snugly against the transparent cover.

6.4 Installing and Replacing the Batteries

- 1) Insert 3 new AAA batteries according to the "+" and "-" for first use, ensure the batteries are aligned properly as indicated inside the battery compartment.
- 2) Replace with 3 new AAA batteries when the flashing battery symbol appears on the LED screen. To change the batteries, slide open the battery cover and remove the batteries.

⚠ CAUTION: Remove the batteries from the product if not used for extended periods of time in order to avoid damage to the nasal aspirator resulting from a leaking battery.

⚠ WARNING:

- Exhausted batteries are to be removed immediately from the appliance.
- Replace all batteries at the same time.
- Clean the battery contacts and those of the equipment with a soft cloth dipped in medical alcohol before battery installation.
- If you think batteries might have been swallowed or placed inside any part of the body, seek immediate medical attention.
- Dispose of used batteries immediately. Keep new and used batteries away from children.
- When there is leakage of batteries on the skin, rinse immediately with clean water and seek immediate medical attention.


Only discard empty batteries. They should not be disposed of in the household waste, but at appropriate collection sites or at your retailer.

6.5 Operating Instructions

Please observe the baby's nasal fluid thickness before use, then choose the right BPA-free suction nozzle. Use the narrow nasal tip for thicker mucus and the wide nasal tip for thinner or larger volumes of mucus.

Start Up

- a) In standby mode, press the power button to start up, everything will be displayed on the screen, and enter the AUTO operation mode by default.

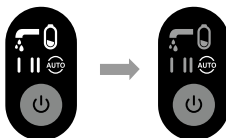Gear Shifting

Press the power button to shift gear, and choose the appropriate gear.

Mode I

Mode II

AUTO


The device offers three selectable suction levels: AUTO (automatic), Mode I (low suction) for gentle use and Mode II (high suction) for more intensive removal of mucus. If the two suction modes prove insufficient due to blockages or significant load fluctuations, it is recommended to switch the device to AUTO mode to maintain effective performance. AUTO mode uses Pulse Suction Technology, controlled by an intelligent chip that rapidly alternates between suction and brief pauses, creating a smooth, wave-like airflow. This produces a gentle, rhythmic effect that helps loosen mucus before removing it efficiently with minimal discomfort. When increased resistance is detected, the device operates in a pulse cycle (3 seconds high speed, 1 second low speed) to ensure effective extraction even in cases of thicker secretions.

Aspirating Mucus

Please choose a comfortable posture and gently insert the BPA-free suction nozzle into the corresponding part of the nose for suction.

Notes:

- 1) The BPA-free suction nozzles should be in full contact with the mucus and the angle between the BPA-free suction nozzles and the nostril should be about 20–30 degrees;
- 2) When the suction nozzle is inserted into the nostril, do not block the other nostril and make sure the baby's mouth is open to avoid negative pressure on the head, causing a pressure imbalance in the ear.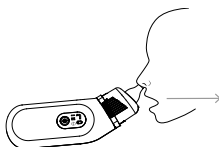

Make sure the mouth is open

- 3) During use, it is recommended that the baby sit up, or half lie down and raise the baby's head, which can achieve the best collection capacity of the collecting reservoir. To avoid damage to the nasal cavity, do not shake the baby's head.
- 4) When the liquid is about to enter the gap between the transparent cover and the side wall of the collecting cup, it is recommended to clean the collecting cup and the transparent cover in advance. If the liquid enters the side wall gap and triggers the electrode to enter the alarm mode, the electrode and its surrounding must be

cleaned.

- 5) When liquid is sucked into the body along the side gap of the collecting cup, it will trigger the electrode to enter the alarm mode: flash prompt “”.

Shut Down

1) Manual shutdown

Press and hold the power button until there is no light indication display on the screen, indicating that the nasal aspirator is turned off.

2) Automatic shutdown

When the nasal aspirator is in the working state and if no buttons are operated for about 65 seconds, it will automatically shut down.

Remove the collecting reservoir

You can keep the BPA-free suction nozzles with product body upwards, and remove the transparent cover and the collecting cup in reverse order by referring to the installation instructions.

Cleaning and disinfection

It is recommended to clean and disinfect the device after each use, and to disinfect before use to avoid breeding bacteria and infection.

a) Cleaning of BPA-free suction nozzles, transparent cover and collecting cup

Remove the transparent cover and collecting cup from the host machine in turn, and then remove the BPA-free suction nozzle when cleaning the transparent cover. It is recommended to clean it with clear water (neutral solvent can be used if necessary). After cleaning, fully dry it and store it in a dry and clean place.

b) Cleaning of the body surface

Wet a soft cloth with clean water and then wring it out to wipe the case of the body. Turn off the power supply when cleaning the product.

If the electrode on the upper part of the machine body has touched the liquid, wipe it with wet or dry paper towels or soft cloth until the indicator light “” no longer flashes (if it is still flashing, the electrode is not completely clean and dry).

c) Disinfection of the Product


The outside of the product is disinfected by wiping the body with a medical alcohol cotton or a soft cloth dipped in 70% to 75% medical alcohol, and then drying with a dry cloth.

d) Disinfection of the BPA-free suction nozzles, transparent cover and collecting cup

Clean the BPA-free suction nozzles, transparent cover, silicone cover and collecting cup with clean water (neutral solvent if necessary), then immerse in boiling water of 99 °C to 100 °C for about 2–3 minutes, take out and dry in a cool place; or clean with clean water (neutral solvent if necessary) and steam

in 95 °C to 97 °C water vapor for about 10–15 minutes, take out and dry in a cool place.

Note: to avoid the liquid entering the vacuum pump from the air inlet and damaging the product, it is forbidden to immerse the body in water or wash it under running water.

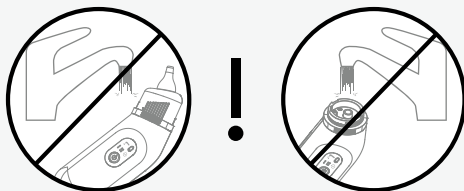7. Troubleshooting

Abnormal problem	Cause	Processing method
No display when you press the power button. 	The battery may be running out if it has been out of use for a long time or there is no battery in the product.	Please replace or install the batteries.
Suction ineffective.	Mucus is dry.	Humidify the dry mucus with saline or sesame oil and suck again 5 minutes later.
	Wrong angle of use.	Try to adjust the angle during the process of suction.
Low voltage indicator. 	Low battery.	Power off your nasal aspirator and replace the batteries, or continue to use until the battery runs out and it shuts down automatically.

Liquid overflow.

The liquid has overflowed from the collecting cup.

Cut off the power, open the transparent cover, clean the collecting cup, and use it after drying. **Clean the electrode.**

8. Care and Maintenance

8.1 Product Maintenance and Storage

- 1) The nasal aspirator should be cleaned immediately after every use. Before use, it should also be disinfected to avoid breeding bacteria and causing infection.
- 2) If the nasal aspirator is not used for a long time, please remove the batteries before storing it, and clean it well.
- 3) It is recommended to do the maintenance at least once every 3–6 months, that is, disinfect, install the batteries and let it work for a while.
- 4) Carefully protect the surface of the display.
- 5) Place the nasal aspirator in a dry place to avoid dust, pollution and direct sunlight.

8.2 Storage and Transportation Conditions

- 1) Ambient temperature: -20 °C–55 °C
- 2) Relative humidity: ≤ 93 %
- 3) Atmospheric pressure: 62 kPa–106 kPa

WARNING:

- 1) Do not modify this equipment without the authorization of the manufacturer.
- 2) The device can not be dismantled by the operators. If reassembly is required, please contact the manufacturer.

9. Product Type

- The product is internally powered equipment.

9.1 Intended Use

The product which is a portable, hand-held, reusable, battery-powered suction device, is intended to enable an adult to gently suction and clear excessive mucus from the nasal passages of an infant or child to facilitate easier breathing in home care.

9.2 Lifespan

The lifespan of the product is 5 years.

10. Important Safety Information

- The warning symbol and other symbols mentioned in this user manual give guidance on safety and the correct method of use, and to avoid injury.
- The warning symbol and other symbols as follow:

Warning symbol	Meaning
	Caution

Symbol	Meaning	Symbol	Meaning
	This device complies with Part 15 of FCC (Federal Communications Commission) Rules.		Model number
	Type BF applied part		Restriction of Hazardous Substances
	Serial number		CE Marking of Conformity
	Consult instructions for use or consult electronic instructions for use		Paper
http://www.mediblink.com/f/m405.pdf			Recyclable
	Date of manufacture		Importer
	Manufacturer		Distributor
	Batch number*		Number of products included in packaging
	Product reference		
	The product and accessories and the packaging have to be disposed of waste correctly at the end of usage. Please follow local ordinances or regulations for disposal.		
IP22	IP22: The first number 2: Protected against solid foreign objects of 12.5 mm and greater. The second number 2: Protected against vertically falling water drops when enclosure tilted up to 15°.		

*The production date can be read from LOT number [YYYYMM]; first four digits represent the Year and last two digits Month of production. Example: LOT 202503 = March 2025

This product is only suitable for suctioning nasal secretions. Please do not use it for other purposes.

Keep the suction nozzle clean and sanitary when in use.

Only use recommended batteries. Do not use rechargeable batteries. To avoid damage to the product, do not mix old and new batteries.

This product should be stored at room temperature. Remove the batteries from the product if not used for extended periods of time in order to avoid damage to the nasal aspirator resulting from a leaking battery.

Keep the nasal aspirator away from sunlight or high temperature and humidity, and keep it in a clean dry place.

Please store this product in the ambient temperature range of -20 °C to 55 °C. When the equipment is transferred from a lower storage ambient temperature to the ambient temperature of 20 °C, it takes 40 minutes for it to return to normal operation. It takes 40 minutes for normal operation to be reinstated from the highest storage ambient temperature to 20 °C.

During storage, please pay attention to the influence of lint, dust, light (including sunlight), regularly use a wet rag to wipe the external surface of the product, and avoid direct sunlight during storage.

During storage, please keep the product away from pets and children to avoid damaging the product or swallowing. Please pay attention to prevent the contamination of this product by pests.

Please do not service or maintain any parts of the Electric nasal aspirator while in use.

If you find any problem, please stop using the device and contact customer service. Self-repair is prohibited.

When the product cannot be used or discarded, please follow the local regulations to dispose of it and avoid environmental pollution.

Only the special accessories provided by the manufacturer can be used with the nasal aspirator.

To prevent liquid from entering the pump and affecting device performance, do not clean the electrode or the air inlet near the pump while the device is in operation.

When using this product, please keep the mouth open, do not hold your breath or swallow saliva, so as to avoid unbalanced ear pressure.

It is strictly forbidden to clean or soak the body parts under the nasal aspirator collection container. Do not immerse the device in water or other liquid, and do not rinse it under running water.

Do not drop, disassemble, repair or transform the device.

Keep away from strong electrostatic or strong magnetic fields, so as not to affect product performance and normal use.

Microwave disinfection and autoclave disinfection are prohibited.

Please read this manual carefully before use. Incorrect use of this product may cause the following symptoms:

- a) Nasal allergy;
 - b) Bleeding of nasal mucosa.
-

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the Competent Authority of the Member State in which the user and/or patient is established.

Please keep the nasal aspirator out of the reach of children under 12 years old. Take special care to avoid inhalation or swallowing of small parts or accessories.

Do not put the nasal aspirator into the fire.

High suction level is not recommended for infants or for first-time use.

Microwave disinfection and autoclave disinfection are prohibited.

Do not share the suction nozzle to prevent cross-contamination.

Do not insert the suction nozzle deeply into the nostril.

Stop immediately if discomfort or crying occurs.

Not recommended for infants under 1 month without medical advice.

Do not use if there is nasal injury, bleeding, or ear infection.

CAUTION:

This unit has been thoroughly tested and inspected to assure proper performance and operation!

11. Package

- 1 nasal aspirator
- 2 BPA-free suction nozzles
- 2 silicone sealing rings
- 1 user manual
- 3 AAA alkaline batteries

WARNING:

The use of ACCESSORIES, cables other than those specified, with the exception of transducers and cables sold by the MANUFACTURER of the nasal aspirator as replacement parts for internal components, may result in increased EMISSIONS or decreased IMMUNITY of the ME EQUIPMENT or ME SYSTEM.

12. Disposal

At the end of its life, the product should not be disposed of in household rubbish. Enquire about the options for environment-friendly and appropriate disposal. Take local regulations into account.

13. EMC Guidance

 **WARNING:** Do not use near active HF surgical equipment and the RF shielded room of an ME system for magnetic resonance imaging, where the intensity of EM disturbances is high.

 **WARNING:** Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and other equipment should be observed to verify that they are operating normally.

 **WARNING:** Use of accessories, transducers, and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.

 **WARNING:** Portable RF communication equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 12 in (30 cm) to any part of the equipment, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

14. Warranty

Product: Mediblink Electric nasal aspirator SuperEasy M405

Manufactured for (Importer for EU and Distributor): Mediblink d.o.o., Gubčeva cesta 19, 8210 Trebnje, Slovenia; info@mediblink.com; www.mediblink.com

Seller's name, address, signature and stamp*:

Date of extradition/sales*:

*If the invoice is accompanied by this warranty, and if all above information can be seen from the invoice, it is not necessary to fill in this field.

Warranty terms

Dear customers!

The warranty period is **3 years** and starts from the day of product purchase. The warranty period on spare parts: Wide suction nozzle, Narrow suction nozzle, Sealing rings (1 large, 1 small), Collecting cup with silicone cover and Transparent cover is **1 year**. In case of product claim, you have to show the invoice. We kindly ask you to save the invoice!

Unfortunately, wrong handling with the device is a reason for 95% of customer complains. You can easily avoid any problem, by getting useful information provided by our special service department. To reach our service department, you can call or send e-mail to Mediblink local Distributor.

Before sending the product back to retailer, we kindly ask you to call our service department, to get help about how to use the device to save you with unneeded trips.

The Manufacturer guarantees free elimination of all imperfections due to defects in material or manufacturing procedure by repairing or replacing the product. In case that the product cannot be repaired or replaced, the customer will get the money refund. The guarantee is not valid in case of the force majeure, accidents or unexpected events (such as lightning, water, fire etc.), incorrect use or incorrect transport, non-compliance with safety and maintaining regulations or in case of unprofessional product intervention.

Traces of every day product usage (scratches, abrasions) and not subject to claim. The warranty does not eliminate the customer rights, which originate from seller responsibility for product flaws. By accepting the claimed product by the service department, the service department does not take responsibility for loss of saved

data or settings on the product. All product repairs, which are performed out of product warranty period, have to be paid by customer by prior notice.

The Manufacturer guarantees the product quality and flawless product operation in the warranty period, which starts with the day of product purchase. If the product cannot be repaired in 45 days, the product will be replaced with a new one. In case that the product cannot be replaced, the money will be refunded to the customer.

In case of product claim, call or send e-mail to Mediblink local distributor.

Replacement parts can be ordered at www.mediblink.com.

Manufacturer:

Hetaida Technology Co., Ltd.
Room 801,802, 803, 804, 901,
2# Building Scientific Research Center,
Songhu Intelligent Valley, No.6 Minfu
Road, Liaobu Town, Dongguan City,
Guangdong Province, P.R.China
E-mail: tomchen@hetaida.com.cn

Manufactured for (importer
for EU & Distributor):

Mediblink d.o.o.,
Gubčeva cesta 19,
8210 Trebnje, Slovenia;
info@mediblink.com;
www.mediblink.com

Instructions for
use version: V.01
Issue date: 22. 03. 2022
Date of last change:
20. 06. 2025

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