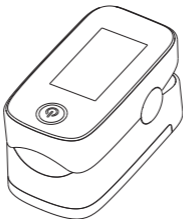
Mediblink[®]
medical devices

EN

Mediblink Pulse Oximeter M170

INSTRUCTIONS FOR USE

Please read the instructions for use
carefully before using the product

1. Product Introduction and operation Guide

1.1 Front View

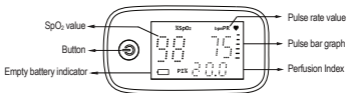
Figure 1: Front View of M170

1.2 Operation Method

- Open the battery cover, and put the two AAA batteries into the battery compartment in correct polarities, then put the cover back in place.
- Press the bottom of the equipment and open the probe, then insert one finger into the probe.
- Press the button to turn the equipment on, and the measure interface will appear.
- After about 8 seconds, the measurement result can be read directly from the display screen.
- Before reading the parameters, make sure that stable numbers of the pulse oximeter interface were sustained for more than 4 seconds.
- The equipment will turn off automatically within 8 seconds after the finger was put off the probe.

1.3 Battery Installation

- A. Put the two AAA batteries into battery compartment in correct polarities (Figure 2).
- B. Push the battery cover horizontally in direction of the arrow as shown on battery cover.

WARNING

- Insert the battery according to correct polarity.
 - Remove the batteries if product will not be in use for a long time.
-

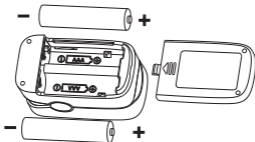
Figure 2: Battery Installation

1.4 Wrist strap Installation

- A. Pass the thinner end of the wrist strap through the hanging hole.
- B. Pass the thicker end of the wrist strap through the thinner end, and tighten the wrist strap (Figure 3).

Figure 3: Wrist strap Installation

1.5 Attention for Operation

- A. Before use, check and confirm that the people or finger size are applicable.
- B. Before use, check and confirm that there are non-combustible materials in the environment, avoid high or low temperature and humidity, and pay attention to the following:
 - a) to avoid glare and direct sunlight exposure;
 - b) to avoid radiation, infrared or ultraviolet radiation;
 - c) avoid contact with the organic solvent, mist, dust, corrosive gases.
- C. The equipment should not be used at a location of limb tied with arterial canal, or near blood pressure cuff, or when receiving intravenous injection.
- D. The equipment may not work normally on micro-circulation barrier patients. Warming or rubbing the finger, or re-positioning the equipment could improve the measurement.
- E. The ray between photo detector and light emitting diode should cross patient's arteriole.
- F. The patient should not use enamel or other makeup.
- G. Avoid inserting a wet finger into the probe.

H. In case of SpO₂ value being below 90%, consult healthcare professional.

Notes:

- A. The user should fully insert the finger into the probe (Figure 4).
 - B. It is recommended to let the LED light shine directly on the nail.
 - C. Don't shake the finger and try to keep the patient still during the measurement.
-

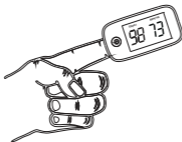
Figure 4: Finger Placement Diagram

! WARNING

- Do not use the oximeter during MRI scans, for periods longer than 30 minutes on the same finger or on neonates and infants.
-

1.6 Functions and Menu Operation

1.6.1 The Button Operation Rules

Long-press functions include: entering menus, activating item's submenu, confirming setting values,

and exiting item's submenu; Short-press functions are polling menu items and viewing the setting values of items. It should be noted that long-press means pressing the key for about 2 seconds, and short-press means pressing the key for less than 0.5 second.



1.6.2 Menu Operation

Activate the menu

After the oximeter is turned on, long-press the power button to activate the menu, then short-press the button to view the setting values of each item. If the user wants to change the setting value of the item, long-press to enter the item's submenu, and the parameter value starts to flash, short-press to traverse the parameter value until the parameter value required by the user is selected, long press to confirm and exit the submenu.

Item 1: Setup the LED Display Brightness

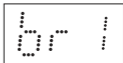
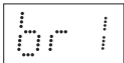
The first item is to setup the display brightness. Long-press the button to select a brightness level ranging from 1 to 3. The greater the value, the greater brightness of the display.

Item 2: Setup the SpO₂ Alarm Limits



The second item is to setup the SpO₂ alarm limits. For example: When SpO₂ High limit is set to 96, an alarm

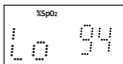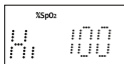will be issued when the SpO₂ value is higher than 96, and when SpO₂ Low limit is set to 94, an alarm will be issued when the SpO₂ value is lower than 94.



Item 3: Setup the PR Alarm Limits

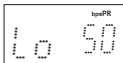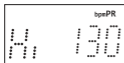The third item is to setup the PR alarm limits. For example: When PR High limit is set to 130, an alarm will be issued when the PR value is higher than 130, and when PR low limit is set to 50, an alarm will be issued when the PR value is lower than 50.

Item 4: Turn Alarm On/Off

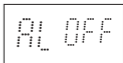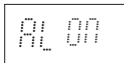The fourth item is long-press to turn Alarm On/Off.

Item 5: Check the software version

The fifth item is to view the software version.

2. Specifications

2.1 Classification

- Type of protection against electric shock:
II (Internally powered equipment)
- Degree of protection against electric shock:
Type BF - Applied part
- Operating mode: Spot checking
- Degree of protection against hazards of explosion:
IP22

2.2 Power Requirements

- Specification of battery: Two AAA (LR03)
- Operating current: 25-50 mA

2.3 Physical Specifications

- Width x Height x Depth: 57 x 30 x 31 mm
- Net Weight: 28 g (product without batteries)

2.4 Measurement Specifications

- SpO₂ declared accuracy: 70%~100%: ± 2 digits
0%~69%: unspecified
- SpO₂ Display Range: 30%~99%
- SpO₂ Resolution: 1%
- PR declared accuracy: 25-250 bpm: ± 3 digits
- PR Resolution: 1 bpm

2.5 Environmental Specifications

Temperature

Operating: +41~+104°F / +5°~+40°C

Storage/Transportation: 40~+140°F / -40°~+60°C

Humidity

Operating: 10~95%, noncondensing

Storage/Transportation: 10~95% noncondensing

Atmosphere Pressure

Operating: 70-106 kPa

Storage/Transportation: 50~107.4 kPa

2.6 Display

Display Type: 1.5" LED Display

Display Color: White

Display Content: SpO₂%, Pulse Rate, PI%,
Bar Graph, Battery Indicator

Notes:

1. The claim for oxygen saturation accuracy should be supported by clinical studies covering the entire claimed range. The fraction of inspired oxygen (FiO₂) delivered to test subjects is varied to achieve a series of targeted steady-state saturation periods over the specified SpO₂ accuracy range (e.g. 70% to 100%), then the SpO₂ accuracy is calculated by comparing SpO₂ readings of the pulse oximeter to the values of SpO₂ determined with a Co-Oximeter.

2. The clinical trial included 11 subjects, including 6 males and 5 females, with an age range of 18 to 46 years, the subjects skin color included dark black, medium black, light color and white.
-

3. Maintenance, Cleaning, Disinfection

3.1 Maintenance

The equipment's design life expectancy is about 2 years, keep your equipment and accessories free of dust and dirt and follow these rules:

- A. Please clean the equipment before use, according to chapter 3.2. Remove the batteries inside the battery cassette, if the equipment will not be operated for a long time.
- B. Replace the batteries in time when the battery voltage indicate lamps were empty.
- C. It is recommended that the equipment should be kept in a dry environment with no corrosive gases and good ventilation anytime. The moisture and high-light environments will affect its lifetime and even might damage the equipment.
- D. It is best to preserve the product in a place where the temperature is between -20 to 60°C and the relative humidity is less than 95%.

- E. The packed equipment can be transported by ordinary conveyance. The equipment must not be transported mixed with toxic, harmful, corrosive materials.

WARNING

- No modification of this equipment is allowed.
 - Keep away from fire and water.
 - Keep out of reach of children.
-

3.2 Cleaning

Your equipment should be cleaned on a regular basis. If there is heavy pollution or lots of dust and sand in your place, the equipment should be cleaned more frequently. Before cleaning the equipment, consult your hospital's regulations for cleaning the equipment.

Recommended cleaning agents are:

- a) mild soap (diluted),
- b) ethanol (70%).

To clean your equipment, follow these rules:

- a) shut down the pulse oximeter,
- b) clean the display screen using a soft, clean cloth dampened with a glass cleaner,
- c) clean the exterior surface of the equipment and probe using a soft cloth dampened with the cleaner,
- d) wipe off all the cleaning solution with a dry cloth after cleaning if necessary,
- e) dry your equipment in a ventilated, cool place.

To avoid damage to the equipment, follow these rules:

CAUTIONS

- Always dilute according the manufacturer's instructions or use lowest possible concentration.
 - Do not immerse part of the equipment in the liquid.
 - Do not pour liquid onto the equipment or accessories.
 - Never use abrasive materials (such as steel wool or silver polish), or erosive cleaners (such as acetone or acetone-based cleaners).
 - If you spill liquid onto the equipment, contact us or your service personnel.
-

3.3 Disinfection

Clean the pulse oximeter before disinfecting it. The recommended disinfectant is 70% ethanol. Disinfection steps are the same as cleaning.

WARNING

- Never use ETO or formaldehyde for disinfection.
-

3.4 Disposal

Dispose of the pulse oximeter in accordance with local environment and waste disposal laws and regulations.

4. Accessories

- Wrist strap
- Two AAA batteries
- User manual

5. Troubleshooting

Trouble	Possible Reason	Solutions
The device can not be turned on	The batteries are drained away or almost drained away.	Replace batteries
	The battery installation is incorrect	Install the battery over again.
	The device works abnormally	Please contact the product distributor
The SpO ₂ and PR are not displayed normally	The finger size is too big or small	Select the suitable size finger to measure
	Excessive ambient light	Avoid the excessive ambient light irradiation
	User's blood perfusion is very low	Warm the finger and try again

Trouble	Possible Reason	Solutions
The display is off suddenly	The device was set to shut down automatically in 8 seconds when there is correct physiological signal	Normal
	The battery is almost drained away	Replace batteries
The SpO ₂ and PR are not displayed stable	The finger is not inserted deep enough	Replace the finger and try again
	The finger is shaking or the body is moving	Try to keep still
	Not used in the work environment required by this manual	Please use in normal working environment
	The device works abnormally	Please contact the product distributor

6. Warranty

Product: Mediblink Pulse Oximeter M170

Manufactured for (Importer for EU & Distributor):

Mediblink d.o.o., Gubčeva cesta 19, 8210 Trebnje,
Slovenija, info@mediblink.com, www.mediblink.si

Sellers name, address, signature and stamp*:

Date of extradition/sales*:

* If the invoice is accompanied by this warranty, and if all above information can be seen from the invoice, it is not necessary to fill in this field.

WARRANTY TERMS

Dear customers!

The warranty period is **2 years** and starts from the day of product purchase. In case of product claim, you have to show the invoice. We kindly ask you to save the invoice!

Unfortunately, wrong handling with the device is a reason for 95% of customer complains. You can easily avoid any problem, by getting useful information provided by our special service department. To reach our service department, you can call or send e-mail to Mediblink local Distributor.

Before sending the product back to retailer, we kindly ask you to call our service department, to get help about how to use the device to save you with unneeded trips.

The Manufacturer guarantees free elimination of all imperfections due to defects in material or manufacturing procedure by repairing or replacing the product. In case that the product can not be repaired or replaced, the customer will get the money refund. The guarantee is not valid in case of the force majeure, accidents or unexpected events (such as lightning, water, fire etc.), incorrect use or incorrect transport, non-compliance with safety and maintaining regulations or in case of unprofessional product intervention.

Traces of every day product usage (scratches, abrasions) and not subject to claim. The warranty does not eliminate the customer rights, which originate from seller responsibility for product flaws. By accepting the claimed product by the service department, the service department does not take responsibility for loss of saved data or settings on the product. All product repairs, which are performed out of product warranty period, have to be paid by customer by prior notice.

The manufacturer guarantees the product quality and flawless product operation in the warranty period, which starts with the day of product purchase. If the product can not be repaired in 45 days, the product will be replaced with a new one. In case that the product can not be replaced, the money will be refunded to the customer.

In case of product claim, call or send e-mail to Mediblink local distributor.

Any serious incident that has occurred in relation to the device should be reported to the Manufacturer, company Mediblink d.o.o. and the Competent Authority of the Member State in which the user and/or patient is established.

Replacement parts can be ordered at www.mediblink.com.

Manufactured for (Importer for EU & Distributor):

Mediblink d.o.o., Gubčeva cesta 19, 8210 Trebnje,
Slovenija, info@mediblink.com,
www.mediblink.com

Instructions for use, version No: V1.1

Issue date: 17. 04. 2020

Date of last change: 04. 02. 2022

K6, 26. 9. 2022

Manufacturer:

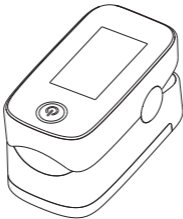
Shenzhen Yimi Life Technology Co., Ltd
305, Building A, Tengbo Industrial Park
Changshangjiang Street, Longbei Village
Pingshan District, 518118 Shenzhen
People's Republic of China

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Shanghai International Holding Corp.
GmbH (Europe)